


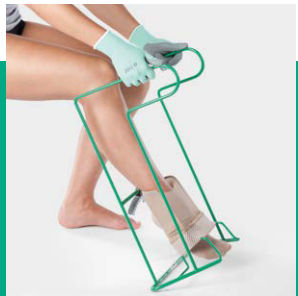
Juzo Easy Fit

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



Donning and doffing aid

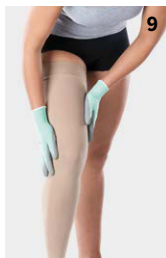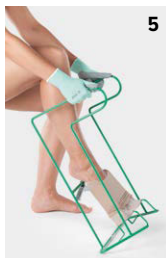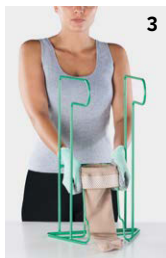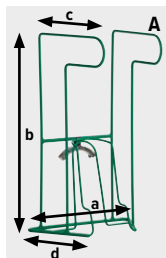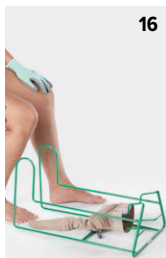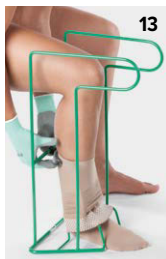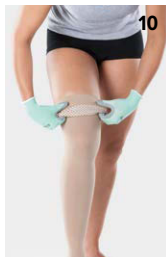Gebrauchsanweisung · Instructions for use · Mode d'emploi · Istruzioni d'uso · Instruções de utilização · Instrucciones de uso · Gebruiksaanwijzing · Brugsanvisning · Bruksanvisning · Käyttöohje · Bruksanvisning · Οδηγίες χρήσης · Kullanım talimatları · Upute za uporabu · Instrukcja użycia · Инструкция по эксплуатации · Návod na používanie · Navodila za uporabo · تعليمات الاستخدام

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EN

Intended purpose

Juzo Easy Fit is a donning and doffing aid for medical compression stockings. The donning and doffing aid is particularly suitable for people with restricted mobility. It not only simplifies the process of putting on and taking off your medical compression stockings, but also gives you additional peace of mind: by using it, you avoid strenuous bending, and the risk of falling is reduced. The donning and doffing aid is also gentle both on your skin and the knitted fabric of your compression stocking. It stretches the stocking beforehand to make it easier to put it on. And the stocking clasp simplifies the process of taking it off.

Notes for use

The Juzo Easy Fit donning and doffing aid is particularly suitable for compression class 1 and 2 phlebological compression stockings. Your specialist retailer will be happy to show you the correct way to use the product. You can also visit juzo.com/donning for video instructions relating to all Juzo donning aids. Put your compression garment on the decongested leg in the morning right after getting up. **Important:** wear Juzo special gloves to protect the fine knitted fabric from damage caused by finger nails,

rings etc. Toe nails and calluses can also cause damage, as can the defective inner lining of shoes.

Using Juzo Easy Fit to put on your product

1. Fold the clasp back and down.
2. Pull the stocking straight. The toe should be pointing downwards (do not turn the stocking inside out).
3. Using both hands, hold the stocking at the top and roll the edge of the stocking over the shaft. The toe of the stocking should be pointing away from the donning and doffing aid.
4. Now roll the stocking over the shaft until the seam of the heel is visible at the edge of the shaft.
5. Now sit down and hold the donning and doffing aid handles at a slight angle. Make sure that your foot is dry. Starting with your toes, push your foot into the foot part of the stocking until your heel is no longer visible.
6. Slide your foot into the foot part until your toes are visible underneath the shaft. Now make sure the donning and doffing aid is almost vertical: raise your toes and apply pressure on the heel at the same time. This rolling motion will help your foot glide into the stocking with ease. Now firmly plant your foot on the floor. **Tip:** Practise this motion sequence several times. It will make donning easier.
7. Using the donning and doffing aid, pull the stocking up to knee-height. Carefully push the donning aid downwards and out of the stocking.
8. Ease the stocking towards your thigh bit by bit with both hands. **Caution:** never pull on the upper edge of the stocking when you do this! Fold over the silicone border (if applicable) before pulling up the stocking. **Important:** wear Juzo special gloves to protect the fine knitted fabric from damage caused by finger nails, rings etc.
9. Distribute the knitted fabric evenly up your leg, smoothing it out with gentle strokes as you go, and avoid creasing.

Using Juzo Easy Fit to take off your product

10. Hold the stocking with both hands and pull it down towards your knee. **Caution:** never pull on the upper edge of the stocking when you do this! Fold over the silicone border (if applicable) before pulling down the stocking. **Important:** wear Juzo special gloves to protect the fine knitted fabric from damage caused by finger nails, rings etc.
11. Now sit down and place your leg into the donning and doffing aid so that your calf is touching the upper curved edge of the shaft. Ease the top of the stocking down

over the shaft until the top of the stocking is level with your heel.

12. Push the entire stocking down as far as the shaft will permit.
13. Using both hands, fold the clasp up towards the shaft until it clicks into place. The stocking is now secured.
14. Place the donning and doffing aid on the floor. Hold both handles with your hands and, for additional stabilisation, put the other foot on one of the lower bars. Raise the foot with the stocking to shaft height. The heel should be above the clasp.
15. Now pull your foot out of the stocking horizontally, parallel to the floor. Avoid pulling the foot up at an angle.
16. Open the clasp by folding it downwards. Now you can remove the stocking easily from the donning and doffing aid.

Safety notice

The donning and doffing aid should be dispensed by properly trained staff only, at a medical retailer. Please contact your specialist retailer in the event of damage. Do not repair the product yourself – this may affect its quality and medical effectiveness. Only use the donning and doffing aid when sitting. Carefully read the instructions for use and follow them. Keep the instructions in a safe place. The manufacturer does not accept any liability for damage/injuries resulting from improper use or misuse. Store Juzo Easy Fit away from children. Only ever use Juzo Easy Fit if the frame and the clasp are in perfect condition.

Cleaning

Wipe the product with a damp cloth and conventional household cleaner.

Material and technical data

Technical data

Item number: 1890

Dimensions (see Illustration A):

- a) 32.0 cm
- b) 52.0 cm
- c) 28.5 cm
- d) 18.0 cm

Frame material: Metal, coated

Clasp material: PVC

Indications

Paralysis, Age-related reduction in strength, Osteoarthritis/rheumatism, Morbid obesity, Extensive stiffening of the spine/hip/knee, Degenerative disorders of the hands/in the area of the hand, Consequence of injuries/amputations

Relative contraindications

Absolute immobility

Liability for failure to comply with this contraindication cannot be accepted by the Julius Zorn GmbH Group.

Side effects

There are no known side effects when this product is used correctly. However, if you notice any adverse changes during the prescribed use, please consult your doctor or medical retailer immediately. In the event of complaints related to this product, please contact the medical retailer directly. Only serious incidents that may lead to a major deterioration in the patient's medical condition or to death should be reported to the manufacturer and the relevant authority in the Member State. Serious incidents are defined in Article 2 (65) Regulation (EU) 2017/745 (MDR).

Disposal

As a rule, the product is to be destroyed via the conventional disposal route after the end of use. Please observe the applicable local/ national regulations.

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Julius Zorn GmbH · Juliusplatz 1 · 86551 Aichach · Germany · juzo.com