

sigvaris



Melany

Donning and doffing device

Melany

Donning and doffing support for everyday use with compression stockings

If mobility and agility are limited, the new Melany donning and doffing device from Sigvaris can help with everyday stocking routines.


Due to its innovative construction, the compression stocking glides effortlessly over the ankle and grips the foot. The effort to bend to fit the compression hosiery is greatly reduced.

What benefits does our Melany donning and doffing device offer you?


- Allows the donning and doffing of stockings quicker.
- Less effort required and therefore less physical strain
- Simplified donning and doffing increases independence
- Reduces wear and tear of compression stockings due to gentle donning and doffing allowing them to retain their therapeutic compression for longer
- Is easy to take apart and stow away for travel or general storage.

How to use Melany for donning and doffing

Donning

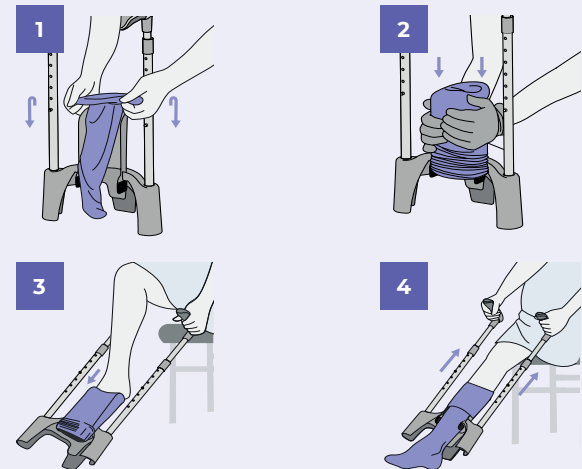

Doffing

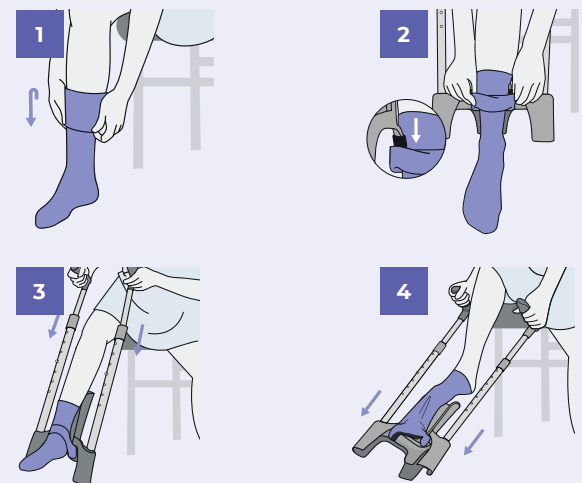

Scan the QR Code to view our Donning and Doffing 'How to' videos




sigvaris

How to measure for Melany

Please follow this simple instruction to determine the correct size.

Measurements should be taken as early in the morning as possible when swelling is minimal.

Step 1	Measure the calf circumference at the fullest part (cC). This should be roughly 25cm up from the ankle bone.
---------------	--

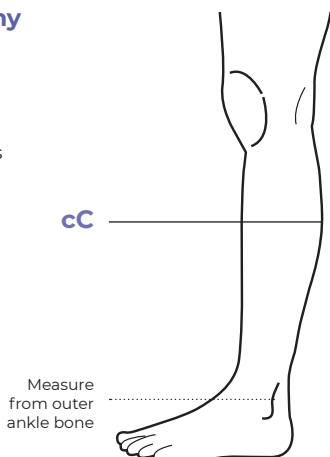

Sizing chart (sizing is shown in centimetres)

	XS	S	M	L / L Plus
Calf (cC)	25-30	30-38	38-46	46-56
Product Code	309386	309387	309388	309389

SIGVARIS Britain Ltd

1 Imperial Court,
Magellan Close,
Andover, Hampshire, SP10 5NT

01264 326666

www.sigvaris.co.uk

With more than 150 years of expertise within medical compression wear, **SIGVARIS GROUP** is dedicated to help people feel their best. Every day



Find us on social media
Sigvaris Group Britain

SIGVARIS
GROUP