



On


Scan the QR code
to watch the video
instructions




The donning aid for closed-toe medical compression stockings and tights with a unique magnetic system

Arion Magnide® provides an easy, safe and simple solution for all wearers of medical compression stockings and tights with a closed toe or their caregivers. Thanks to the products' unique sliding properties and magnet system, you can now put on and take off closed-toe compression hosiery with ease. Arion Magnide® has a flexible sizing to fit different limb sizes. This is particularly important for patients diagnosed with lower limb edema (swelling).

Before you start:







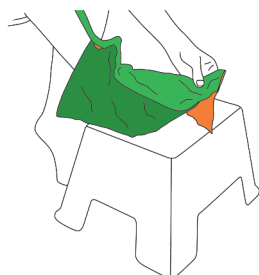


Important!
Don't push the foot to the
very end of the Magnide®,
leave some free space
at the end

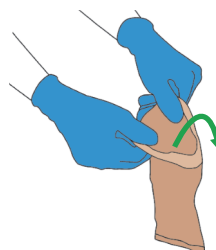


In case the compression
garment has a silicon border,
make sure to turn it outwards

Donning instructions

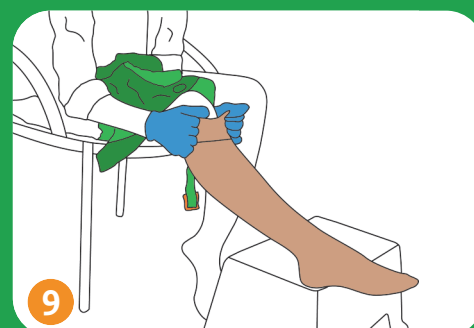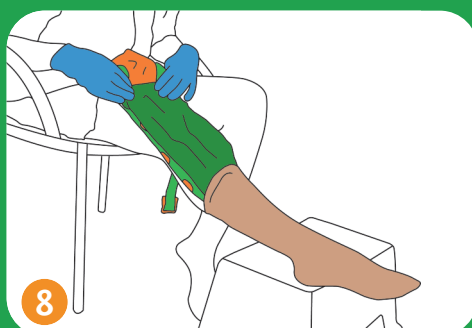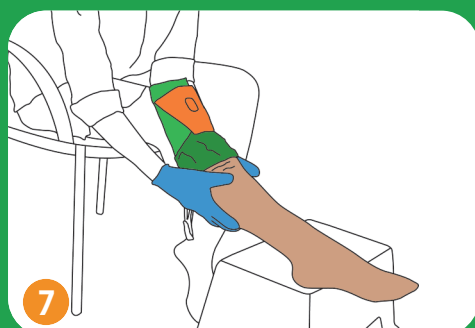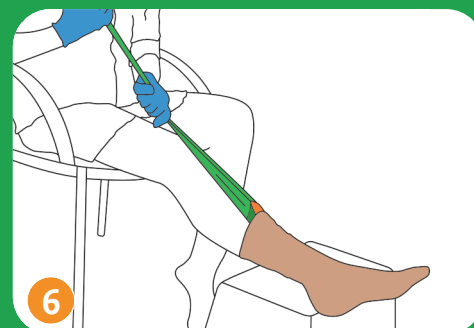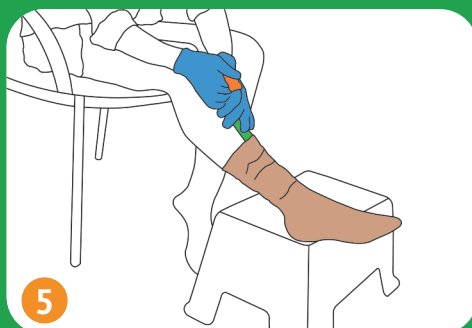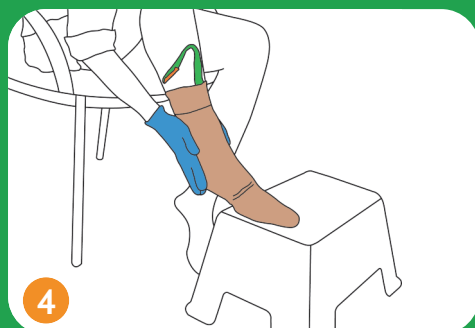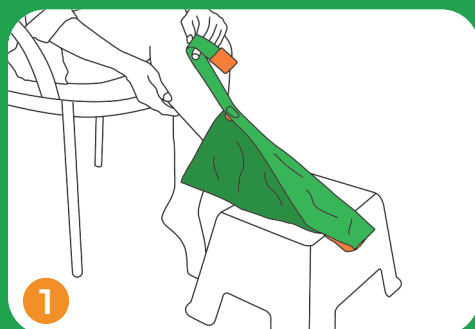

Fix the top of the loop to the top of the Magnide® by using the magnet

1

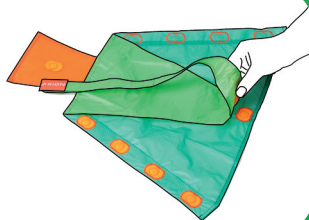


Turn over the Magnide® and place it on a flat surface

2




Fold both green sides to the middle. The orange magnet tabs must meet each other and make contact.

3

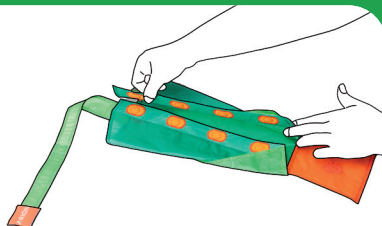

Close the orange flap (near the toes) over the Magnide® so that the magnets make contact

4

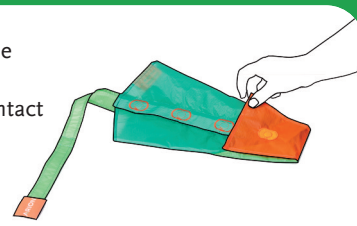

Compression therapy made easier

Arion donning and doffing aids for medical compression garments

Easy to Use

- Minimize friction between compression garment and skin
- Reduce physical strain for both compression wearers and caregivers
- Promote independence of compression wearers

Safe to Apply

- Reduce risk of skin damage ; can be especially beneficial in case of high skin sensitivity or leg ulcers
- Contribute to a longer garment lifetime by preventing damage from fingers and nails
- Help maintaining the original compression longer and avoiding garments from overstretching and distortion

Simple Solutions

- Application aids for all types of compression garments: stockings, tights, arm sleeves, and gloves
- Designed for different shapes and sizes of limbs
- Support the concordance to compression therapy

Arion Magnide®

SIZES

Art. Nr.	Size	Shoe Size EU	Shoe Size (UK)	
			Woman	Man
E01513	Medium	≤ 38	≤ 5,5	≤ 6
E01514	Large	39 - 43	6 - 8	6,5 - 9
E01515	X-Large	≥ 44	≥ 9	≥ 9,5

XL also suitable for extra wide feet
(circumference up to 11"/28 cm)



Should you need another size, Arion can customize the product according to your needs.