

GUS COMFORT

Application and Removal Aid for Hosiery



INSTRUCTIONS INSIDE

PUTTING ON YOUR GARMENT USING GUS COMFORT


 **Caution:** Sharp fingernails, rings and bracelets can damage garments. Rings and bracelets should be removed before you apply/remove your garment.



- 1** Attach the Velcro at the toe end to form a triangle.



- 2** Place Gus on the floor in front of you and place your foot in the opening.

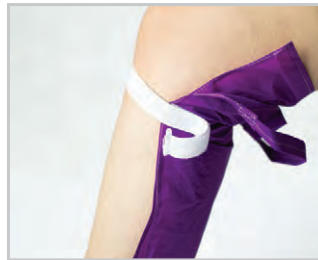

- 3** Attach the white fixing strap around the leg just above the calf



- 4** Pull your stocking on over your toes and then slowly roll it up your leg.



- 5** Unfasten the fastener at your toes. (This may not be necessary as Gus opens quite easily by itself.)



- 6** Undo the fixing strap. Slowly and gently, using the handle at the back pull Gus from under your stocking.

Note: The use of donning gloves and/or a non-slip mat can aid in the application of compression garments

MAINTENANCE / STORAGE

- Store your Gus in a cool, dry place
- After washing, make sure Gus is completely dry before further use
- When not in use keep the Velcro portions attached, to avoid build up of fluff
- Gus Comfort should not be used beyond the expiration date printed on the outside of the package.

TAKING OFF YOUR GARMENT USING GUS COMFORT

 **Caution:** Sharp fingernails, rings and bracelets can damage garments. Rings and bracelets should be removed before you apply/remove your garment.



- 1** Place foot between two halves of the Gus with the strap at the back. Pull the Gus up to below the knee and secure the Velcro Strap around the leg.



- 2** Fold stocking over the strap.



- 3** Slide the stocking off over the Gus, easing it over the ankle/heel area

GUARANTEE

GUS products are made from quality materials and are manufactured with modern techniques. When used according to the instructions for use the period of warranty is 6 months.

ADVANTAGES


- Easy to use
- Easy to clean
- Durable construction
- One size fits all
- Suitable for any kind of compression stocking

GUS COMFORT



DISTRIBUTED BY:

Haddenham Healthcare Ltd
Crendon House
Crendon Industrial Park
Long Crendon
Bucks HP18 9BB

 01844-208842

 sales@hadhealth.com

 www.hadhealth.com



Composition

90% ripstop nylon | 10% polyester

PIP-CODE: 360-0285

LOT HAD001007UK2013A

 2017-12



HMC-NEDERLAND | HERMELIJNLAAN 22, 5581 CJ WAALRE | THE NETHERLANDS

