

# Credalast

## COMPRESSION

	SMALL	MEDIUM	LARGE	EX-LARGE
Thigh	40–54cm	46–60cm	54–68cm	60–74cm
	(15_"–21_")	(18"–23_")	(21_"–26_")	(23_"–29_")
Calf	28–34cm	32–38cm	36–42cm	40–46cm
	(11"–13 <u>"</u> )	(12_"–15")	(14_"–16_")	(15_"–18")
Ankle	18–22cm	20–24cm	24–28cm	28–32cm
	(7"–8_")	(8"–9_")	(9_"–11")	(11"–12_")

Standard Length	Thigh Length	Up to 65cm (25_")	
	Below Knee	Up to 38cm (15")	
Long Length	Thigh Length	Up to 80cm (30")	
	Below Knee	Up to 45cm (18")	

### **SELECTING THE CORRECT SIZE**

#### Circumference Measurements

Measure round the ankle at narrowest point above the ankle bone.

Measure round the widest part of the calf.

Measure round the thigh at the position the stocking will reach on the leg.

### **Length Measurements**

Below Knee: Base of heel to just below knee.

Thigh: Base of heel to where stocking will finish on leg.

### **INDICATIONS FOR USE**

CLASS 1	Superficial or early varices. Varicosis during pregnancy.
CLASS 2	Varices of medium severity. Ulcer treatment and prevention of recurrence. Mild oedema. Varicosis during pregnancy.
CLASS 3	Gross varices. Post thrombotic venous insufficiency. Gross oedema.  Ulcer treatment and prevention of recurrence.

#### **CREDALAST NYLON**

	STYLE	TOE	FIBRE CONTENT	SHADE
Class 1	Below Knee	Full Foot	88% Nylon 12% Lycra	Natural
Class 1	Below Knee or Thigh	Open Toe	75% Nylon 25% Lycra	Natural
Class 2	Below Knee, Thigh or Thigh + Belt	Open Toe	70% Nylon 30% Lycra	Natural
Class 3	Below Knee or Thigh	Open Toe	70% Nylon 30% Lycra	Natural

#### **CREDALAST COTTON**

	STYLE	TOE	FIBRE CONTENT	SHADE
Class 1	Below Knee	Open Toe	22% Lycra 47% Nylon 31% Cotton	Natural
Class 2	Below Knee	Open Toe	22% Lycra 47% Nylon 31% Cotton	Natural
Class 3	Below Knee	Open Toe	22% Lycra 47% Nylon 31% Cotton	Natural
Class 1	Thigh Length	Open Toe	24% Lycra 38% Nylon 38% Cotton	Natural
Class 2	Thigh Length	Open Toe	24% Lycra 38% Nylon 38% Cotton	Natural
Class 3	Thigh Length	Open Toe	24% Lycra 38% Nylon 38% Cotton	Natural

Thigh at widest point stocking will reach on leg

Widest point of the calf

Ankle at narrowest point above the ankle bone